

- Your VertaLoc® suspension sleeve is guaranteed to be free of material or workmanship defects.
- Any defective VertaLoc® suspension sleeve may be replaced or credited at the discretion of VertaLoc, Inc.
- This product is under warranty for one year from the date of purchase.

Questions or comments?

WE ARE ALWAYS HERE TO HELP YOU.

Call us toll-free:

855-VERTALOC (855-837-8256)

WARNING: This product is not designed to prevent injury. If you experience skin irritation, pain or discomfort while using this product, remove immediately and contact your healthcare provider.

SUSPENSION SLEEVE Patient Guide

Model# VL310SS



VertaLoc, Inc. P.O. Box 1271, New York, NY 10159

www.vertaloc.com

Caring for and Storing your VertaLoc® Suspension Sleeve:

Hand wash as needed in cold water with mild detergent. Rinse thoroughly and air-dry out of direct sunlight. Do not machine wash. Do not twist and squeeze. Do not expose to direct heat sources such as heaters, stoves, and radiators.

- ✓ Do not store the suspension sleeve in a hot (80°F +) or humid (80% +) area.
- ✓ Store the wrap in a cool, dry place (heat may cause suspension sleeve deformation).
- ✓ Keep the suspension sleeve away from dust or pollutants.

FOR SINGLE PATIENT USE ONLY

Page 4

THANK YOU FOR USING THE VERTALOC® SUSPENSION SLEEVE.

This easy-to-use wrap was designed to enhance comfort, protect skin and prevent rashes.

YOU HAVE RECEIVED:

- One (1) Suspension Sleeve

INSTRUCTIONS FOR USE:

1

Open the suspension sleeve and wrap around the knee with neoprene side facing the skin.



Page 1



2

Gently press Velcro strap against fabric to close the sleeve.



Page 2

The **VertaLoc® Suspension Sleeve** fits comfortably under all **VertaLoc®** knee braces to enhance knee brace's comfort and protect your skin from chafing.



Page 3