

THANK YOU FOR USING THE VERTALOC® SHOULDER BRACE

This easy to use shoulder brace was designed to maximize support and enhance comfort while targeting a wide variety of diagnoses.

(MODEL# VL711SB)
HCPCS L3960

YOU HAVE RECEIVED:

- ONE (1) SHOULDER ORTHOSIS BRACE COMPLETE WITH THE CLOSURE SYSTEM **(REVERSIBLE FOR LEFT OR RIGHT SHOULDER)**
- ONE (1) NECK STRAP
- ONE (1) WAIST STRAP
- TWO (2) PLASTIC WEDGES
- ONE (1) REMOVABLE PALM PAD



← Removable “black square Velcro” attached to the side fabric of the brace.

FIT YOUR BRACE IN A FEW SIMPLE STEPS

1. Attach the shoulder strap to the brace. Slip the **shoulder strap** over the neck. Adjust the length of the strap if needed. Attach the “black square Velcro” to the side fabric. Attach the waist strap to the “black square Velcro” and bring the strap around your body. Insert the end of the strap with Velcro through the loop; pull and fasten the waist strap for a secure fit.



WAIST STRAP

2. Close each arm strap with the **Velcro connectors**. Two wedges are included with the shoulder brace. Each wedge is used for a different angle of abduction. One wedge allows shoulder positioning at 15 degrees; another – at 30, 60, or 90 degrees. Place one of the wedges with the desired angle of abduction under the armpit.



3. Reverse the arm straps Velcro connectors to use the brace for the opposite shoulder.

WARNING:

Your healthcare provider is familiar with your medical condition for which this brace was prescribed. Always follow your physician's directions on how to use this device.

CARING FOR YOUR SHOULDER BRACE:

Hand wash fabric components in warm water using mild soap. Rinse thoroughly. Air-dry out of direct sunlight. Do not expose to direct heat sources such as heaters, stoves, and radiators.

VERTALOC® SHOULDER BRACE WARRANTY:

- Your VertaLoc® shoulder brace is warranted to be free of material or workmanship defects.
- Any defective components of your VertaLoc® shoulder brace may be replaced or credited at the discretion of VertaLoc, Inc.
- This product is warranted for one year from the date of purchase.

QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.
CALL US TOLL FREE 855-VERTALOC (855-837-8256)